Vivago

Vivago MOVE™

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Vivago MOVE™ solution operates in the background of an Android phone, the user will not need to open the solution after it has been installed. The solution can send notifications, in case the watch or the solution show disconnections.

Please note! The use will be interrupted, if the Vivago MOVE™ solution is removed or closed, or if the Bluetooth or data connection are not in use.

The Vivago MOBILE solution is meant for monitoring wellbeing information and for receiving alarms. The Vivago MOBILE solution is supported by Android, iPhone and Windows phones.

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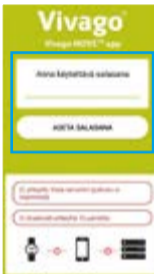
Preparations:

1. Charge the watch.
2. Take out your order confirmation.
3. Activate the Bluetooth connection on your phone and make sure that the data connection is working.
4. Install the Vivago MOVE™ app on Google Play.
5. Wear the watch in order to activate it.
6. Make sure the phone and watch are close to one another to ensure connection and real time transfer of the data.
7. Please also read the user's guide available on guides.vivago.com

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1. Open the application

Create a password by typing it in the field. Use this password for signing in to the application.

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3. Allow location permission

Only when the location is allowed, the user can be located. The location permission can be denied later by changing the settings.

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4. Choose the settings view

Add the Vista server's address and port. You can find the information on your order confirmation. Save the information.

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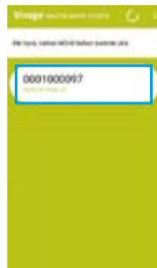
5. Choose the status view

Press the Couple MOVE watch button. Please note! You will have to wear the watch in order to couple the watch with the application.

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6. Choose the MOVE watch ID number

Choose your MOVE watch. Note! Make sure, that the application number corresponds to the number on the back of the watch or the one on the order confirmation.

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7. Launch the background service

In the status view, please press the Launch background service button. Note! Make sure that the phone and watch are close to one another to ensure connection and real time transfer of the data.

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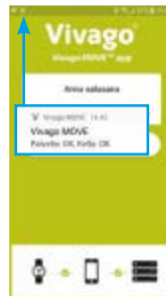
8. Make sure that the connection is working

The connection is working, if you can see two green symbols between the watch and phone icons in the status view of your application. You can now start using the watch. A red cross between the watch and phone icons, or between the phone and server icons indicates, that there is no connection established yet.

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Now the Vivago MOVE™ watch is ready for use You can access the status information by swiping down from the top of the phone screen. The opening view indicates the server and watch status. The application opens by choosing this view.

Please note! For monitoring the wellbeing information and for receiving alarms you will need to download the Vivago MOBILE application, which can be done in Google Play Store or App Store. You will find the registration information on your order confirmation.

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CM29007-03

Important!

If you feel that you need help, press immediately the VIVAGO button on the watch until a red light on the watch appears. The red light will be switched on a while after the alarm has been sent. **Do not wait for the system to send an automatic alarm for you!**

You can send an alarm yourself by pressing the alarm button on the watch. The watch sends an automatic alarm e.g. in case of immobility within 30 minutes up to 4 hours.

- Vivago MOVE™ watch measures user's activity, sleep and circadian rhythm.
- The user can use the watch and mobile application for calling for help at home or outside.
- Care providers and relatives can keep track of user's wellbeing information.
- Automatic notifications and alarms will signal any changes in user's wellbeing.

Wear the watch on your non-dominant hand. Fasten the watch on your wrist so that it feels comfortable and is not restricting mobility. It has to sit tight enough, so that the bottom of the watch is in close contact with the skin even when moving the arm around.

The watch should be kept on the wrist all the time since the watch continuously monitors the activity level from your wrist.

Do not open the watch. The warranty will be void in case the watch is opened.

Basic precautionary measures should always be taken into account, including the below:

- If the instructions in the user manual are not followed, the device may not function correctly.
- Do not wear the watch in hot environment, e.g. in a sauna.
- Do not charge the watch battery in a hot environment.
- Do not use the watch during a swim or bath.
- The watch should not be left in direct sunlight or with children or pets.
- Do not charge the watch, if the case is broken.
- Do not charge the watch, if the case is broken.

The watch is not intended for making treatment decisions or for replacing professional medical advice, nor used by children younger than 3 years.

Save these instructions!
Please also read the user's guide
[guides.vivago.com](https://www.vivago.com/guides)

Functions

The Vivago MOVE™ watch has three buttons:

- 1 The **VIVAGO** button on the watch is meant for triggering alarms, but also for confirming changes in the setting mode of the watch.
- 2 A long press on the **LIGHT** button will switch on the background light. The light will automatically switch off after 12 seconds.
- 3 The **MODE** button is for browsing through the different menus on the watch.

Settings

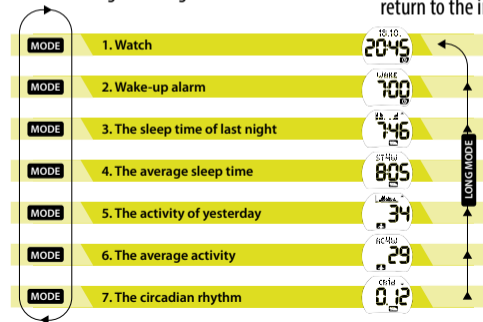
By pressing LIGHT(2) and MODE(3) buttons simultaneously, you will enter the settings mode. The setting mode depends on in which display mode you enter the settings.

In the set-up mode the buttons work as follows:

LIGHT: Changes settings.

MODE: Changes numbers, for instance hours to minutes in time settings, or changes the settings.

VIVAGO: Confirms the setting and moves you to the next set-up mode. If you don't press any button for 60 seconds you will automatically return to the initial watch mode.

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CM29007-03

Vivago® develops smart safety and wellbeing solutions for preventive care. The innovative solutions cover the whole chain of care from home care to assisted living, care homes, hospitals and rehabilitation.

Vivago's solutions increase the feeling of safety with automatic alarms and by providing wide range of wellbeing information to support daily care and to follow-up the effectiveness of care. Real-time notifications inform about changes in the person's wellbeing and it enables care providers to react proactively and focus on individual care.

Vivago Ltd, Itsehallintokuja 6, FI-02600 Espoo, Finland, www.vivago.com – move.vivago.com/en/

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